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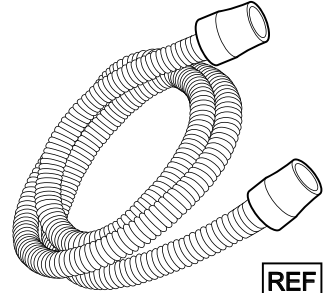
# BREAS

## Patient Circuit, Z1 Slim Style CPAP Tube

REF

005797

Breas Medical AB

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SE-435 33 Mölnlycke

SWEDEN

Phone: +46 31 86 88 00

Fax: +46 31 86 88 10

Web: www.breas.com

UK Responsible Person:

Breas Medical Ltd.

CE 2797 UK CA 0086

## User instruction

### Z1 Slim Style CPAP Tube

#### 1 Intended Use

The intended use of the Z1 Slim Style CPAP Tube is to deliver air from the CPAP device to the patient. The tube is intended for single patient use. The tube is intended for home and hospital use.

#### 1.1 Contraindications

Do not use the Slim Style CPAP Tube in cases where CPAP treatment is contra-indicated.

Please refer to the Instructions for use of the CPAP device.

#### 2 Safety Information

Read this manual so that you completely understand how the patient circuit is operated before taking it into use, to ensure correct usage and maximum performance. Also read the manuals for the devices you connect the patient circuit to.

Do not sterilize or disinfect the patient circuit. Clean as described in this manual only.

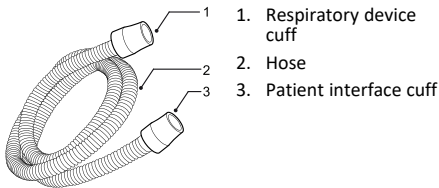
Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority.

#### 2.1 Usage

This circuit is for single patient use only. Reuse to new patients may cause cross-infections.

Appropriate personnel should determine the duration of use based on accepted infection control procedures. Replace the circuit if it becomes discoloured, cracked or damaged.

#### 3 Product Description

1. Respiratory device cuff
2. Hose
3. Patient interface cuff

#### 3.1 Connecting and Disconnecting the Patient Circuit

Connect and disconnect the tube by handling the cuffs. Do not pull the tube. For detailed information about connecting to the ventilator, see the ventilator's user's manual.

#### 3.2 Cleaning and Maintenance

The patient circuit should be cleaned as described once a week. All parts that come into contact with the respiration gas shall be cleaned. Cleaning is only for

continued use by the same patient, do not reuse a patient circuit to a new patient.

- 1 Wash the patient circuit in lukewarm water with a mild detergent or soap solution.  
Do not use disinfectants, antibacterial soaps, scented solutions or oils to clean the tube. These may create a shock hazard and reduce the life of the tube.
- 2 Rinse thoroughly.
- 3 Hang the tube to dry. Do not use the patient circuit before it is fully dry. Check for discolouration, damages or cracks.

#### 3.3 Disposal

The patient circuit must be disposed of and recycled in accordance with the local environmental regulations regarding the disposal of used equipment and waste.

#### 3.4 Technical Data

Property	Specification
Standard compliance	ISO 5367:2014
Patient Category	Adult and Paediatric
Length	1.2 m.
Resistance	cmH <sub>2</sub> O/l/min @ l/min
Compliance	ml/cmH <sub>2</sub> O @ cmH <sub>2</sub> O
Inner diameter	15 mm
Max recommended working temperature	43 °C (109.4 °F)
Max working pressure	60 cmH <sub>2</sub> O
Storage	Dust-free. Protected from sunlight. Prolonged exposure to sunlight may cause the tube to discolor and degrade.

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